

Durasphere®
Injection Needle
Directions For Use

Contents:

- 1 Injection Needle

Injection needle is available in a variety of sizes.

Directions for Use

- Using a standard procedure, prepare the patient for injection.
- Connect the injection needle to the Durasphere syringe. The injection needle must be tightened securely to the syringe.
- Advance the injection needle to the desired location for injection.
- Upon completion of the procedure, withdraw the injection needle.

Warning: After use, the product may be a potential biohazard. Handle accordingly and dispose of in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

- **Sterile unless package has been opened or damaged.**
- **Single Use Only.**
- **Do Not Re-sterilize.**

Do not use opened or damaged packages; return to Coloplast Corporation for replacement. Do not use if packaging is not intact or if the injection needles appear to be damaged.

Product should be at room temperature, 60°F - 90°F, prior to use.

CAUTION: Federal law restricts this device to sale by or on the order of a physician.

Durasphere is a registered trademark of Carbon Medical Technologies, Inc.
United States Patent Number 6,565,550.

Manufactured and distributed by: Carbon Medical Technologies, Inc.
1290 Hammond Rd.
Saint Paul, MN 55110 USA
Customer Service: 1-888-207-0262

Order from: Carbon Medical Technologies, Inc.
1-800-207-0262

QTY	Quantity
LOT	Batch code
REF	Catalog number
	Use by date
	Manufacturer and Date of manufacture (YYYY-MM-DD)
	Consult instructions for use
	Do not use if package is damaged
	Do not re-use
	Do not re-sterilize
	Single sterile barrier system
STERILE EO	EO Sterilized
	Distributor
R_x Only	By prescription
	Caution